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# Lightweight Wheelchair: Instruction Manual

Article No.: KREHA-2080-45

## Foreword

Dear Customer!

Thank you for purchasing a wheelchair from REHASHOP. The carefully designed REHASHOP wheelchair is light, comfortable and efficient. It is flexible and safe to use, and enjoys great popularity among the majority of its users. Before use, please read these instructions in order to better understand the functions of the wheelchair and therefore better control it. You should carry out maintenance and servicing as necessary to ensure the wheelchair is in good condition. For any questions, please contact the dealer or the manufacturer and we will be happy to help!

## Assembly and Adjustment

### 1.1 Adjusting the angle of the footrests:

Use the 6mm hex key to loosen the screws on the side of the footrests. Now you can adjust the angle between the footrests and the floor, and then retighten the fixing screws.

### 1.2 Adjusting the height of the footrests:

Unscrew the knobs on the rear of the tubes by hand and remove the bolts from the front. Now you can adjust the height of the footrest to fit the size and leg-length of the user. Once the foot supports are at the right height, insert the bolts into the pre-drilled holes and retighten the knob.

#### Attention

The angle of the footrests can only be adjusted when they are at least 5cm from the floor.

**1.3 Pivoting the leg supports:** Press the lever on the leg supports towards the seat and then pivot them outwards.

To reposition the leg supports in front of the wheelchair, press the lever once more and move the footrests back into their original position.

#### Attention

Please always check before moving off that the footrests are correctly locked into place.

## 1.4 Removable armrests:

### 1.4.1 Folding up the armrests:

Press the forward lever on the armrest to release the catch on the frame and enable the armrests to be folded up.

### 1.4.2 Removing the armrests:

Press the forward lever on the armrest to release the catch on the frame and enable the armrests to be folded up. Now press the rear lever to completely remove the armrests.

### 1.4.3 Adjusting the height of the armrests:

Press the large button on the side of the armrest with one hand and with the other hand, position the armrest at the desired height. Release the button and make sure that the armrest locks into position.

### 1.4.4 Adjusting the horizontal position of the armrests:

Press the little button which is located directly on the armrest to enable the armrest to be moved forwards and rearwards. Make sure that the armrest locks into position again.

## 1.5 Adjusting the angle of the front wheels:

Use the 5mm hex key to remove the wheel mount. Now position this at the desired angle and retighten the screws.

## 1.6 Adjusting the height of the front wheels:

Remove the fixing screws and nuts from the front wheels. Now choose one of the three holes to attach the wheels back to the mount. After tightening the screws, check whether the wheels turn freely. If not, you should slightly loosen the screws and nuts, so that the wheels can turn easily.

## 1.7 Adjusting the height of the rear wheels:

First, remove the wheels from the frame. These can be easily removed at the touch of a button (on the locking mechanism in the centre) thanks to the quick-release axles. Now remove the wheel mount and its fixing with the help of the tool. You can now reattach both components in one of the holes in the wheelchair frame. The wheels must then be reattached. To do this, press the locking button and push the wheel back onto the quick release axle. Make sure that the rear wheels lock into position on the quick release axle.

### 1.8 Adjusting the horizontal position of the rear wheels:

First, remove the wheels from the frame. These can be easily removed at the touch of a button (on the locking mechanism in the centre) thanks to the quick-release axles. Now use the 5mm hex key to remove both screws on the top and bottom of the mounting plates. Move the mounting plates forwards or rearwards, taking care to position them over the holes provided for this purpose. The rear wheels must now be reattached to the frame. To reattach the rear wheels, press the button on the locking mechanism and push the wheel onto the axle. Make sure that the rear wheels lock into position on the quick release axle. If applicable, the parking brake must now be adjusted, so that it is still capable of reliably stopping the rear wheel.

### 1.9 Adjusting the anti tip:

The anti tip is fixed with a spring clamp. By pulling the washer, you can adjust the height of the anti tip. When the desired height is achieved, simply release the adjustment lever. Make sure, however, that the anti tip audibly locks into position in one of the holes so that it sits securely in the fixture.

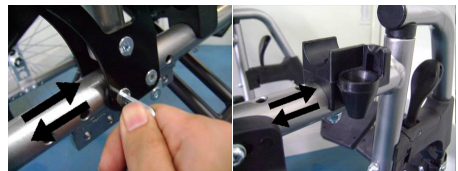
### 1.10 Adjusting the angle of the backrest:

Remove the armrests and rear heels from both sides as previously described. Using the 5mm hex key, loosen the screws which hold the backrest to the frame. You can now adjust the angle of the backrest and then retighten the screws and the nuts.

### 1.11 Adjusting the depth of the seat:

Remove the armrests and rear wheels from both sides as previously described. Using the 5mm hex key, remove both the lower screws on the support which is also connected to the backrest. You can then move the support horizontally forwards or backwards, which in turn changes the seat depth. Using the pre-drilled holes in the frame tube, reinsert the screws and retighten them.

### 1.12 Attaching the cushion:

Press the cushion into place on the outer sides of the wheelchair seat.

## Instructions for use

### 2.1 Opening the wheelchair:

Gently push downwards on the frame at the sides of the seat to unfold the wheelchair. Now smooth out the seat with your palms.

#### Notice

When unfolding, don't pull too hard on the armrests to avoid damaging the components. In order to avoid injuring your fingers, you should be careful when unfolding the wheelchair that your hand doesn't get caught between the frame and seat tubes.

### 2.2 Folding up the wheelchair:

Fold the footrests up. Then take hold of the seat at the front and at the rear under the seat in the middle and pull upwards. If needed you can push together the tubes at the side of the frame to which the seat is attached.

#### Attention

You should only sit down in, or stand up from the wheelchair when the parking brake is applied in order to avoid accidents. When sitting down and standing up, avoid standing on the footrests to avoid accidents.

### 2.3 Sitting down and standing up

#### *Sitting down:*

First, apply the wheelchair's parking brake and fold the footrests up. Walk backwards towards the wheelchair, support yourself lightly on the armrests, and slowly — if necessary with the help of your accompanying person — lower yourself down into

the seat. Next, fold the footrests down and position your feet on them.

#### *Standing up:*

First, apply the wheelchair's parking brake and take your feet off the footrests so that these can be folded up. Support yourself lightly on the armrests and — if necessary with the help of your accompanying person — lift yourself out of the wheelchair.

### 2.4 Travel operation

2.4.1. When travelling outdoors, you must observe the traffic regulations. If the wheelchair is being pushed, the accompanying person must use the tip assist to negotiate obstacles.

2.4.2. When negotiating several steps or larger obstacles, the wheelchair and the user should be held by two people and carefully lifted over the obstacle.

2.4.3. If the wheelchair is pushed down a steep slope, the accompanying person should do this backwards in order to prevent the user from falling forwards out of the wheelchair.

#### Attention

1. Do not brake abruptly while travelling in order to avoid accidents.
2. If you are going down a slope, you should brake very carefully in order to avoid accidents.
3. Do not go down a slope of more than 8° in order to avoid accidents.

## Symbole

This way up

Fragile

Keep dry

## Maintenance

Check the parts and screw fixings of the wheelchair at regular intervals. If any screws have come loose, they must be tightened in order to avoid safety risks. Moving parts should be treated weekly with lubricating oil in order to guarantee their functionality. Use a soft dry cloth to clean the frame of the wheelchair after use in order to prevent rusting.

Keep the cushion and the backrest clean in order to avoid a skin infection. If the wheelchair is not in use, it should be put in a dry well-ventilated location.

Please observe current regulations when disposing of the packaging material from the wheelchair.

## Drum brake – accompanying person

The drum brake is activated by the accompanying person using the brake lever on the handle. The accompanying person can finely regulate the brakes in order to ensure safe travel.

**Function as a service brake:** Lightly and evenly apply both brakes, and the wheelchair will gently slow down.

**Function as parking brake:** To secure the wheelchair against unwanted rolling away, evenly apply both brakes. With a finger, press the safety catch on the brake lever until this locks into place and fixes the brake lever. Then release both brake levers and check whether the bolt mechanism is locked and the wheelchair is secured. The wheelchair must not be pushed when the brakes are locked. To release the drum brake, pull on the brake lever until the safety catch automatically pops out of the locked position. Please release both brake levers. The parking brakes are now released and the wheelchair is ready to move again.

### Notice

If the parking brake displays insufficient braking power, the brake must be adjusted in a professional workshop.

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KAPHINGST ONLINE AG
Proteno GmbH