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Instruction manual lightweight wheelchair

Item No.: KREHA-2070

The "Zebra" lightweight wheelchair is used to transport people in a sitting position, whether in the household, when shopping, or when travelling. The wheelchair is equipped with removable footrests and a holding brake. Additionally, the lightweight wheelchair features a drum brake for the wheelchair user's companion as well as quick-release axles to enable rapid removal of the rear wheels. As the wheelchair can be folded, it can be transported in a car and/or stowed away to save space.

We hope you enjoy using the REHASHOP "Zebra" lightweight wheelchair and hope that it will enable you to gain greater mobility.

- | | |
|----|--|
| 1 | Push handles and handbrake for companion |
| 2 | Armrests |
| 3 | Leg supports |
| 4 | Propulsion wheel |
| 5 | Hand rim |
| 6 | Quick release axle |
| 7 | Tipping protector with tilting aid |
| 8 | Holding brake |
| 9 | Guide wheel |
| 10 | Footrests |

Assembling the wheelchair

Ensure that the wheelchair is complete and assemble the wheelchair in the following order:

- Unfold the frame as shown and describe in the section headed "Unfolding the wheelchair"
- Attach the back and seat cushion to the wheelchair. For a description, see "Removable back and seat cushion."
- Attach the footrests as shown in the section entitled "Attaching and locking the footrests."

Unfolding the wheelchair

Press down on the seat frame. The chair will unfold itself. Now push the seat frame into the supports. This step is easier if you tilt the chair a little, thus lifting some weight off the propulsion wheel. Check that the seat frame is properly engaged with the seat and that the seat is flat.

Folding the wheelchair

First remove the leg supports and the seat cushion. Now grip the centre of the seating surface from below and pull up. The chair will fold up and, if required, the backrest is also collapsible.

Folding backrest

It is possible to reduce collapse the backrest to reduce the height of the wheelchair if required. Push the two black levers on the back of the chair at the same time. The folding mechanism will unlock and the backrest can be folded down.

Important: Ensure that the backrest is fully unfolded and locked in place before using the wheelchair.

Detachable back and seat cushions

Attach both cushions before using the wheelchair. The cushions are folded in place and attached using a Velcro fastener.

Footrests and leg supports

The wheelchair is equipped with swivelling footrests. The footrests can be swivelled upwards to make entry and exit easier.

The leg supports can be also be swivelled outwards and returned to position as a whole. The leg supports are also removable to enable the wheelchair to be folded as small as possible.

The leg supports are swivelled outwards to insert them, then turned inwards with by applying slight pressure until they lock into place. Remove the leg supports by pressing the unlocking button and swivel the footrests outwards.

When attaching the leg supports, always ensure that they are properly engaged.

Important: the leg supports and footrests are not suitable for carrying or lifting the wheelchair when it is occupied.

GB

Armrests

To adjust the armrest height, use the side button to unlock the armrest and set the desired height. Release the button to lock the armrest into place. When travelling in the wheelchair, we recommend that you set the armrests to the highest position.

Important: Side parts are not suitable for lifting or carrying the wheelchair.

Tyres

The REHASHOP "Zebra" lightweight wheelchair is equipped with a full set of puncture-resistant rubber tyres. There is no need to check the tyre pressure. If the tyres are worn or excessively aged, they can be replaced by a specialist retailer.

Detachable axles on the rear wheels

The wheels used to propel the REHASHOP "Zebra" lightweight wheelchair come equipped with a quick release axle as standard, which means that they can easily be removed without any tools. To remove the wheels, press the locking button on the

outside of the axle on the propulsion wheel and pull down on the wheel at the same time. If re-assembling in reverse order, ensure that the locks of the quick release axles are engaged! The locking button must bounce back to its original position. After re-assembly, always check to ensure that the rear wheels are firmly in place!

distance as shown above. Tighten the screw after completing the operation.

Important: If any adjustments are made to the configuration of the rear wheel, the brake configuration must be rechecked and may need to be reset.

Important: If the all-rubber tyres wear out, the effectiveness of the brakes may be affected. The effectiveness of brakes should therefore be checked at regular intervals.

Tilting aid with tipping protector

The wheelchair user's companion can use the tipping tilting aid found at the bottom of the wheelchair to tilt it. When standing behind the wheelchair, the companion steps on the left or right-hand tilting bar making it easy to slide the wheelchair over the kerb.

Holding brakes

The wheelchair is equipped with two holding brakes. The brakes are applied directly to the rear wheels. To hold the wheelchair in position, press both brake levers as far forward as they will go. The holding brake (parking brake) is released by pulling the brake lever backwards.

Brake performance declines when

- the tyre profile is heavily worn
- the tyres are wet
- the brakes are improperly configured.

The holding brakes are not designed to stop the wheelchair in motion, and the brakes should not be operated when the chair is moving.

Always use the brake provided for the companion to brake the wheelchair.

Please note that the gap between the holding brake and the tyre is approx. 3 mm. To adjust the brake, loosen the fastening screw on the frame and set the

Cleaning and Maintenance

Water and, if necessary, a mild detergent can be used to clean the wheelchair. Tougher stains can be removed using car shampoo. When cleaning the wheelchair, no water should be allowed to penetrate the frame. For that reason, the wheelchair must only be cleaned when fully assembled. Servicing should be performed by your specialist retailer when required or replace the wheelchair regularly if it becomes worn out. If regular inspection reveals defects in braking power, contact your retailer immediately! The wheelchair must not be used without being repaired by a specialist! Perform the following maintenance regularly:

- Check the tyres for wear every four weeks.
- Check every six months to ensure that all screws are tightly seated.
- Lubricate the quick release axles using sewing machine oil every eight weeks.
- Seawater, road salt and sand damage the bearings and steering components of the wheelchair.

After using the wheelchair in areas affected by the above, the wheelchair should be cleaned thoroughly using fresh water.

Warranty

We provide a warranty for this product in accordance with European requirements, with the warranty period covering the first twelve months. This warranty is restricted to production and assembly defects. We are not liable for wear and tear or for any improper use or any abuse of the wheelchair. The warranty will be voided in the event of any modifications or repairs to the chair performed by unauthorised retailers or any other unauthorised person. It does not cover any costs incurred for repairing defects, freight charges, loss of use or loss of service, nor for any travel or similar costs. Similarly, damage or injury arising from any improper use, misuse, or failure to follow the instructions are not covered by the warranty. The making of any technical changes to the product is expressly prohibited.

Technical data

Available seat widths: 41, 46 und 51 cm
 Seat depth: approx. 41 cm
 Wheelchair weight: approx. 14.5 kg
 Rear wheels: 24" (approx. 61 cm) in diameter
 Front wheels: 7" (approx. 18 cm) in diameter
 Armrests: height adjustable
 Footrests: removable and height adjustable
 Seat upholstery: black and grey Nylon
 Frame: Powder coated aluminium
 Maximum load capacity: 110 kg

General safety instructions

Only use the wheelchair if it is free from defects. Any inappropriate use of the wheelchair can cause danger to the wheelchair user. For your own safety, you must follow the rules set out below:

- Improperly performed adaptations or customisations increase the risk of accidents.
- As a wheelchair user, you are part of the general road traffic and we therefore draw your attention to the fact that, as a wheelchair user, you are subject to the general road traffic laws.
- Wear the lightest clothing possible during the hours of darkness.
- When using the wheelchair for the first time, take particular care and get used to the effect of changes in the centre of gravity on the handling of the wheelchair.
- Practise using the wheelchair on flat ground, on gradients and on slopes.
- The wheelchair's holding brakes are not suitable for stopping the wheelchair when in motion!

- The holding brakes are designed to prevent the wheelchair from rolling away when stationary. If you stop on an uneven surface, always use both holding brakes. If the brakes are applied only on one side, there is a risk that the wheelchair could tip over.
- Use wheelchair ramps or lifts. If these are not available, you should seek help to reach your destination. The person assisting you should only hold on to firmly attached parts of the wheelchair.
- The wheelchair must not be lifted with its occupant; it must only be pushed if occupied.
- Check the quick release axles on the rear wheels, the Velcro fasteners of the seat and the backrest, the tyres, the tyre pressures if applicable, and the brakes.
- The maximum permitted load of the wheelchair is 110 kg.
- The wheelchair must be used exclusively to transport people. No other use is permitted in line with the stated purpose.
- When used on other forms of transportation, such as trains or buses, the wheelchair user must not sit in the wheelchair while travelling! Only seats fitted in the relevant method of transport offer adequate protection in the event of a dangerous incident.
- If you make any changes to the configuration of the wheelchair, it is important to read the relevant Uneven surfaces can cause the wheelchair to tip over, particularly when it is used on slopes or gradients. If the wheelchair travels forwards up a step or a gradient, the user's body should be tilted forwards.
- The wheelchair must only be used on firm ground.
- The wheelchair is not designed to provide support when standing up out of a squatting or sitting position.
- The wheels of the chair may jam if they encounter an uneven surface, e.g., in gutters or gratings. For this reason, you should avoid obstacles of this nature wherever possible.
- Apply the wheelchair's holding brake when parking the wheelchair to prevent any the wheelchair from unintentionally rolling away.
- Always load the wheelchair vertically from above.
- On wet, dirty, icy, or snow-covered surfaces, there is a significant risk of slipping! The effectiveness of the brakes is also severely limited on these surfaces! Adjust your usage style accordingly.
- Even when all detachable parts have been removed, the wheelchair still weighs more than 10 kg. For this reason, the wheelchair should be lifted

with the push handle and the front tubular frame
in a folded position.

- If you have any problems or queries, please contact your specialist retailer.

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KAPHINGST ONLINE AG
Proteno GmbH