rehashop

REHASHOP Outdoor Rollator Falcon

Art. No.: A005491

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Thank you very much for your confidence in our products.

Please read the following information and instructions carefully before using your new REHASHOP outdoor rollator Falcon.

For a safe handling of your new rollator, please note the instructions for use and safety in particular.

If you have any questions, please contact our customer service team. You can reach them free of charge at the number **0800 888 90 80**

We wish you all the best with your new product!

Your rehashop team

Manufacturer:

CHIEN TI ENTERPRISE,. LTD.

No., Lane 227, Fu Ying Rd., Hsin Chuang District,
New taipei City, Taiwan

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Information from the European representative:

EMERGO EUROPE

Prinsessegracht 20

2514 AP The Hague

The Netherlands

GB

Table of contents	Page
1. Introduction	13
2. Intended use	13
3. Product dimensions and specification	14
4. Assembly instruction	14
5. Handle adjustment.....	16
6. Break operation & adjustment	16
7. Maintenance	17
8. Cleaning	18
9. Warnings	18

Instruction manual

REHASHOP outdoor rollator Falcon/ Article No. A005491

Symbols used in this manual:

Symbol for warnings

Please follow the instructions on the side of this symbol. Ignoring them may result in physical injury or damage to the rollator and your surroundings.

GB

Introduction

The rollator is manufactured from high quality materials and includes seat, bag, locking brakes. In addition, an air pump for the Falcon's tyres is included. The handles are adjustable in height and the rollator folds easily for transport or storage.

Intended use

Designed to assist the user to walk, rest and carry small items of shopping for indoor and outdoor use on flat surfaces by a single individual limited mobility with a maximum weight as shown below. The intended user should possess the necessary physical, visual, and cognitive skills to operate the rollator safely. If you have any doubt, please consult your dealer/prescriber. It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

Do

1. Always ensure the brakes are in the locked position if the rollator is left unattended.
2. Check all castors and wheels before using.
3. Check the brakes before using.
4. Check handle height adjustment hand knobs before using.
5. Check if the rollator is fully unfolded.
6. Take care while carrying the rollator folded.
7. Keep your hands away from the folding mechanism to prevent any risk or pinching.

Do not

1. Use the rollator as an aid to transport persons, it is not designed as a complete wheel chair.
2. Sit on the rollator if the brakes are not locked, this could cause the rollator to move resulting in injury to the user or other persons.
3. "Drop" into the seat of the rollator, always sit into the rollator in a smooth, controlled motion. Seek assistance if necessary.
4. Hang objects on the brake levers or hand grips. This could affect brake operation and/or instability of the rollator. Every care has been taken to ensure that your rollator will give you safety, reliable service provided you adhere to the advice given above.

GB

Product dimensions and specification

Overall Height	Overall Length	Overall Width	Seat Depth	Seating Width	Seat Height		Castor Size	Unit Weight	Capacity
									
Min. 90 cm/ Max. 103 cm	90 cm	66 cm	25 cm	46 cm	60 cm		30,5 cm	11,05 kg	125 kg

1. Backrest
2. Adjustable parking / running brake
3. Seat area
4. Height adjustable handles
5. 350 mm swivel front castors
6. Shopping bag
7. Cane holder

Assembly instruction

The Falcon is delivered to you folded together.

1. Take the rollator out of its packaging.
2. Lay down the Falcon, as illustrated below, with the handles lying on the floor.

GB

3. First, fold down the front wheels, which are now found at the top, and secure them with the help of the enclosed star-shaped screws.

4. Set the rollator upright, on its front wheels.
5. Fold down the rear wheels

6. Secure the rear wheels, by pushing the pins hanging from the tape through the hole provided, and lock them in place with the clamp.
7. Once the rollator is standing stable on all four wheels, slide apart the two handles and push down the seat.

Handle adjustment

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set to the desired height. Repeat this for the opposite side and make sure that both handles are secure and set at the same height before use.

GB

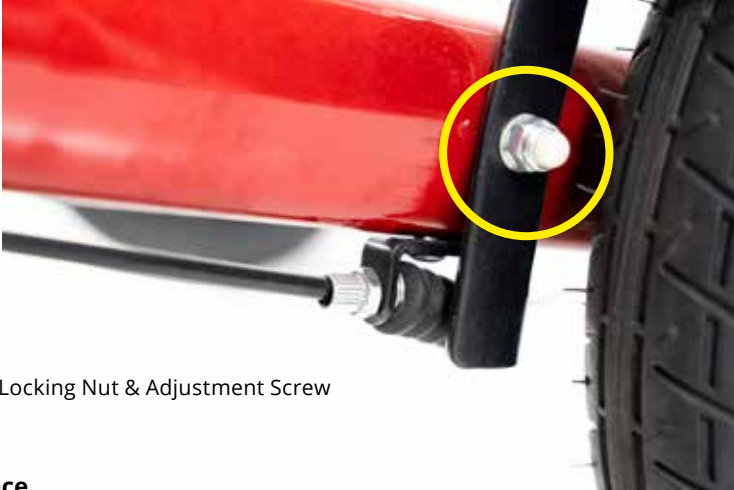
The spring button should be located in the same position on either side.

1. Pull up on brake handle lever to slow the rollator
2. Push down on brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock. Brakes must be applied in the lock position when the seat is going to be used, pre check the lock by pushing forward the rollator.

Brake adjustment is carried out like this:

Turn the adjuster screw anti-clockwise to increase the tension then lock the adjustment by turning the locking nut clockwise while holding the adjuster screw stationary. Brake adjustment reduced to allow light pressure may be ineffective on the brake in the locked position.

Pictured: Locking Nut & Adjustment Screw

Maintenance

Over a period of time the brake cables may stretch, and if no further adjustments can be made the cables should be replaced. Seek the help of your dealer when replacing brake cables.

Castors should be checked that they are secure each time before using the rollator. Also check if the brakes operate correctly and if the handles and hand knobs are secure. Should any part of the rollator appear loose or insecure stop using the rollator until it is inspected by an authorised dealer. The following shows maintenance schedule.

Weekly:

Check brake for malfunction. Check all wheels and castors for loose bolts, nuts and knobs.

Periodically:

Check hand grips for wear. Check if frame bolts are secure. Check if front and rear axle bolts are secure. Check tyres for wear or cracks.

Yearly:

Take your rollator to an authorised repairer for a full check, clean, service and lubrication of all moving parts one a year.

Serviceable components: Wheels, hand grips, brake cables, brake lever, bearings, knobs, bag, bolt, nut.

Cleaning

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleansers may be used if required. Do not leave any cleaning residues on the hand grips or seat. They may be slippery when used. Do not use scouring pads or scrapers because this may damage surface coating. Store your rollator in ventilated and dry place. Avoid oleic acid material to avoid corrosion.

GB

Warnings

1. Maximum working limit 125 kg.
2. Do not stand on the rollator.
3. Ensure that both push handles are adjusted to the same height.
4. Seek advice from a trained health care professional or an approved freedom health care dealer for height adjustment of this product.
5. Always maintain a safe distance between your rollator and other pedestrians or objects.
6. Some components may cause harm or damage when they get into contact with other objects.
7. Do not attempt to replace the wheels or castors of your rollator. If they are faulty or require replacement please contact an authorised repairer.
8. Ensure that your rollator is opened fully (folding scissor arm is straight) before using it.
9. Ensure that your fingers are clear from the scissor arm when folding.

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PROTENNO GmbH
Kaphingst Online GmbH