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CONTI TOILET SEAT RAISER SOFT

PRODUCT DESCRIPTION

Toilet seat raiser made of polyurethane foam – 100% corrosion-proof. Presents no pressure marks after prolonged sitting on the toilet. Tool-free installation – easy to attach to the toilet seat. Smooth surfaces allow easy cleaning. Delivered in practical carrier.

INDICATION

Toilet seat raiser to increase the toilet bowl height. Facilitates sitting down and getting up without assistance. To be used in case of mobility impairment and when use of conventional toilet bowls is impossible.

SAFETY INSTRUCTIONS

Please check the product for possible defects before use. Do not use defective product and contact the manufacturer. Make sure not to exceed the maximum load of 100 kg!

PACKAGE CONTENT

Carrier
Instructions for use
Toilet seat raiser made of PU foam

NOTES

TECHNICAL SPECIFICATIONS

Dimensions

Length 45 cm, width 29 cm

2 different heights

Art.No.: T-1094 Height 5 cm

Art.No.: T-1095 Height 10 cm

Maximum load: 100 kg

Colour: blue

INSTALLATION

Place the toilet seat raiser carefully on the toilet seat. To avoid shifting, the limit stop at the front should be in contact with the inside of the toilet bowl. The toilet seat raiser is simply placed on top of the toilet seat and does not require any screwing on. Please ensure that the toilet seat raiser is not moved during use.

CLEANING

Clean the toilet seat raiser regularly with warm soapy water or mild, non-aggressive cleaning or disinfection agents. Do not use any abrasive/acidic cleaning or scouring agents as they damage the material. We assume no warranty in this case.

MAINTENANCE

Check the product regularly for defects. In case of defects, inform the manufacturer immediately and do not use the product any longer.

WARRANTY AND PRODUCT LIFE

The warranty is 12 months as of the date of purchase. Errors and changes due to improvements in technical design are subject to alterations. If used as intended and according to safety, care and maintenance instructions, the toilet seat raiser has an expected service life of up to 3 years. Further use of the toilet seat raiser beyond this period is possible when the product shows no defects.